

Alcolizer Easy Check

Portable Breath Alcohol Tester



Introduction

The ALCOLIZER EASY CHECK (EC) breath alcohol tester is a portable instrument that is simple to use and gives an accurate and reliable result at a reasonable price. It has a variety of applications and is ideal for the budget – conscious buyer who is seeking quality and reliability. The EC is certified by Australian Standards.

Reliable Accuracy

The EC is designed and manufactured by Alcolizer Technology (AT), an Australian firm with a high reputation as a manufacturer of quality breath testers. The EC uses the same advanced sensor technology used for years in police breath testers worldwide, combined with the latest in microprocessor technology.

Easy to Live With

ALCOLIZER EASY CHECK is small and uses 2 aaa alkaline batteries so that it can be used anywhere. Mouthpieces are supplied, and additional mouthpieces are available. AT provides a 2-3 day turn-around recalibration service – the EC even tells you when it is due!

Easy Operation

Simply push the button and observe the automatic circuit testing. The LCD shows “BLO” when the unit is ready. Just breathe into the mouthpiece for a few seconds until the audio tone signals that the test is complete. The blood alcohol content is indicated on the meter to the nearest 0.001%. The unit powers down to a low power mode after testing to save battery power. Note it does not turn off completely.

The Bottom Line

ALCOLIZER EASY CHECK is an economic solution for personal breath-testing applications.

For further information, please contact:

Alcohol Countermeasure Systems

22/9 Hoyle Ave

Castle Hill NSW 2154

Tel: 02 8850 6251 Fax: 02 8850 7647

E-mail: ascaa@ozemail.com.au

ALCOLIZER EASY CHECK - SPECIFICATIONS

Size:	14.5 cm x 6.8 cm x 2.5 cm
Weight:	200 grams
Accuracy:	better than $\pm 10\%$ (0.010 at 0.10% BAC)
Number of tests:	200 tests or six months between calibrations
Batteries:	2 aaa alkaline
Operation in ambient temperature:	5°C to +40°C (30°F to 100°F)
Purge cycle:	30 seconds
Breath sample:	5-second moderate and continuous breath sample
Readout	Immediate after analysis.
Smoking caution:	Wait 3 minutes
Drinking (alcohol consumption) caution:	Wait 15 minutes
Mouthpieces	Round type
Calibration:	Mail-in calibration service by AT technicians
Display:	LCD with backlight
BAC format:	% BAC selectable by factory
Complete package includes:	EC UNIT Plastic Storage case 4 mouthpieces Operators Manual Warranty Card Return post satchel

Optional Equipment

Spare mouthpieces

Warranty

ALCOLIZER EASY CHECK personal alcohol testers are warranted to be free from defects in workmanship and material for 6 months from the date of shipment by AT Pty Ltd. AT will replace or repair any unit or part that in its judgement is found to be defective, providing that the defect has not been caused by misuse or mishandling. Units in warranty must be properly packaged and shipped prepaid to AT Pty Ltd.