

ALCOLIZER HH1

Portable Breath Alcohol Tester



Introduction

The ALCOLIZER HH1 breath alcohol tester is a portable instrument that is simple to use and gives an accurate and reliable result at a reasonable price. It has a variety of applications and is ideal for the budget – conscious buyer.

Reliable Accuracy

ALCOLIZER HH1 is designed and manufactured by Alcolizer Technology (AT) an Australian firm with a high reputation as a manufacturer of quality breath testers. The ALCOLIZER HH1 uses the same advanced sensor technology used for years in police breath testers worldwide, combined with the latest in microprocessor technology.

Easy to Live With

ALCOLIZER HH1 is small and uses a 9 volt alkaline battery so that it can be used anywhere. Mouthpieces are supplied, and additional mouthpieces are available. AT provides a 2-3 day turn-around recalibration service – the HH1 even tells you when it is due!

Easy Operation

Simply turn on the ALCOLIZER HH1 and observe the automatic circuit testing. The LCD shows “BLO” when the unit is ready. Just breathe into the mouthpiece for a few seconds until the audio tone signals that the test is complete. The blood alcohol content is indicated on the meter to the nearest 0.01%. The unit powers down after testing to save battery power.

The Bottom Line

ALCOLIZER HH1 is an economic solution for many breath-testing applications and is widely used in Australia by industry and individuals. It is flexible enough for any application. For further information, please contact:

Alcohol Countermeasure Systems

22/9 Hoyle Ave
Castle Hill NSW 2154
Tel: 02 8850 6251 Fax: 02 8850 7647
E-mail: ascaa@ozemail.com.au

ALCOLIZER HH1

Portable Breath Alcohol Tester

Specification

Size:	14,5 cm x 6.8 cm x 2.5 cm
Weight:	200 grams
Accuracy:	better than $\pm 10\%$ (0.010 at 0.10% BAC)
Number of tests:	300 tests or six months between calibrations
Batteries:	1 9 volt alkaline
Operation in ambient temperature:	5°C to +40°C (30°F to 100°F)
Purge cycle:	30 seconds
Breath sample:	5-second moderate and continuous breath sample
Readout	Immediate
Automatic power down:	After 30 seconds of inactivity the unit shuts off
Smoking caution:	Wait 3 minutes
Drinking (alcohol consumption) caution:	Wait 15 minutes
Mouthpieces	Round type
Calibration:	Mail-in calibration service by AME technicians
Display:	LCD with backlight
BAC format:	% BAC
Complete package includes:	HH1 UNIT Vinyl Storage case 4 mouthpieces Operators Manual

Optional Equipment

Leather case

Spare mouthpieces

Warranty

ALCOLIZER HH1 personal alcohol testers are warranted to be free from defects in workmanship and material for one year from the date of shipment by AT Pty Ltd. AT will replace or repair any unit or part that in its judgement is found to be defective, providing that the defect has not been caused by misuse or mishandling. Units in warranty must be properly packaged and shipped prepaid to AT Pty Ltd.

Alcohol Countermeasure Systems Australasia Pty Limited

22/9 Hoyle Ave
Castle Hill NSW 2154
Tel: 02 8850 6251 Fax: 02 8850 7647
E-mail: asca@ozemail.com.au